

Manual to the rehabilitation robot

Developers: Tim Verbelen
Abdulrahim Hashim
Jeroen Vander Eeckt

Usage without computer

- Plug the power cable in the outlet
- The screen will indicate the start-up
- There are 2 buttons underneath the screen, with those you can adjust the level

- The levels indicate a certain force the user needs to apply

Level	Pretension (cm)	Starting force (N)	Force end (N)
1	0	0,00	52,00
2	0,26	2,70	54,70
3	0,52	5,41	57,41
4	0,78	8,11	60,11
5	1,04	10,82	62,82
6	1,3	13,52	65,52
7	1,56	16,23	68,23
8	1,82	18,93	70,93
9	2,08	21,64	73,64
10	2,34	24,34	76,34
11	2,6	27,04	79,05
12	2,86	29,75	81,75
13	3,12	32,45	84,46
14	3,38	35,16	87,16
15	3,64	37,86	89,87
16	3,9	40,57	92,57
17	4,16	43,27	95,28
18	4,42	45,97	97,98
19	4,68	48,68	100,69
20	4,94	51,38	103,39

- Pull the handle, and watch the screen indicate off how far you are.
- If you pull the handle far enough the program will adjust the counter +1.
- If you want to reset the counter you need to push the 2 buttons underneath the screen simultaneously for 5 seconds.

Usage with computer

- Plug-in the USB cable in your computer
- Install the program included in the package
- Move the robot; you should see the mouse moving
- The revalidation robot is now a mouse for your computer
- The pulling handle is the left mouse button
- The little button near your thumb is the right mouse button
- Work, play,... whatever you want with your computer, and watch the counter rise trough the roof!